

Instructions For Use For



Oxehealth Seclusion

en English

Table of Contents

Table of Contents	2
Symbols Used In This Manual And On Product Labels	3
Instructions For Use For Oxehealth Seclusion Product	3
Intended Use	3
Contraindications, Warnings and Cautions	3
Warnings	4
Cautions	4
Maintenance	5
How To Use the Seclusion Product	6
The Overview Screen	6
Starting a Session Report	9
Taking Vital Signs Observations	11
Viewing a Session Report	12
Setting Patient ID	14
Seclusion Summary Report	14
Exporting a Summary Report	18
Exporting via the Summary Reports screen	18
Exporting via a Session Report	20
Room Actions for Seclusion Rooms	21
Patient ID	22
Tracking when Rapid Tranquillisation is administered	23
Prompts for Vital Signs Overdue	27
Sending Feedback to Oxehealth	28
Manufacturer	28

Symbols Used In This Manual And On Product Labels

	Read the instructions for use		Caution
---	-------------------------------	---	---------

Instructions For Use For Oxehealth Seclusion Product

Carefully read all instructions prior to use. Observe all warnings and precautions noted throughout these instructions. Failure to do so may result in misunderstanding of the system outputs.

These instructions for use are for the Oxehealth Seclusion product, which is a product module within the Oxehealth Oxevision platform. For instructions on how to use other product modules within the Oxehealth Oxevision platform, refer to other instructions for use for those product modules.



Oxehealth Seclusion uses the Oxehealth Vital Signs medical device, and so the instructions for Vital Signs should be read prior to use.

Intended Use

The Oxehealth Seclusion product is an electronic patient record system that provides a timeline summary of categories of:

- Activity detected by Oxehealth Activity Tracker products,
- Vital sign measurements from the Oxehealth Vital Signs medical device, and
- indications of rapid tranquilisation (RT) events recorded by clinicians.

The Seclusion product is a reporting tool intended to support management of seclusion sessions by recording rapid tranquilisation events alongside data from Oxehealth Activity Tracker and Vital Signs products. The Seclusion product does not have a medical purpose. The Seclusion product is not intended to be used as an independent medication record.

It is for use within single occupancy rooms within secured environments where a framework exists which mandates continuous observation and periodic safety checks by a trained professional to ensure subject safety. The product is designed to provide additional information in parallel to the authoritative human checks.

The Oxehealth system is intended for use by appropriately trained healthcare professionals with a duty of care, and should not be used by untrained users.

The Seclusion product is indicated for use on adults and adolescents¹, who do not require critical care in accordance with the intended use of the Oxehealth Activity Tracker and Vital Signs products.

Contraindications, Warnings and Cautions

Observe all contraindications, warnings and cautions in the Oxevision Activity Tracker and Vital Signs Instructions for Use. Failure to do so may result in harm to patients or users.

¹ 12 years and older, under 18 years



Warnings

The Seclusion product must not be used in rooms that contain or may contain:

- Domestic animals
- A shower or significantly sized water feature.

The Seclusion product is not a patient monitor or a vital signs monitor. Oxehealth Vital Signs provides vital signs spot check measurements. Its outputs do not indicate whether the person it is monitoring is alive, nor does it give any indication of the health or safety of an individual. The product cannot be used to determine whether an individual is dead. It must not be used as the sole basis for making any decisions or recommendations relating to clinical diagnosis, subject care or treatment. Decisions or recommendations must be made by a clinician based on broader evidence than the Seclusion product alone provides.

The Seclusion product is not a medication record or planning tool for tranquilisation of patients. Decisions or recommendations for administering tranquilisation must not be made based on the rapid tranquilisation indications in the Oxehealth Seclusion session report.

Cautions

The Seclusion product may give incorrect information about activity in the room in the following circumstances:

- Large airflows through the room
- Other sources of movement in the image
- If another living being is present in the image (including hidden from view, for example by sheets)
- Running, dripping or spraying liquids in the image
- Moving patches of light or shadow in the image
- In the presence of sources of flashing or variable light, including other Near Infrared illumination, for instance from an existing CCTV installation
- Sources of mechanical vibration, for example massage devices, in the image
- Vibration of the Oxehealth hardware, caused for example by drilling into a nearby structure
- In the event that someone dies but continues to move (for example to swing or to twitch) for a period after death.
- Where insufficient body area is visible to the camera
- If the area the camera sees of the subject is heavily saturated due to exceptionally bright illumination
- In subjects with pronounced sleep apnoea or who are deliberately controlling their breathing
- The subject's torso is covered by a material with extremely low contrast edges
- The subject is in a dark area of the room while a large part of the remainder of the room is covered in direct sunlight.

The product analyses video captured from a video camera within a room illuminated by a Near Infrared Illuminator, and so obstruction of either of these parts of the device will severely compromise its operation.

Oxehealth software and related hardware installed inside or outside the room environment is not to be installed or maintained by anyone other than an Oxehealth trained representative. Any attempt by a non-Oxehealth representative to interfere with the hardware, operating system administration

or software configuration outside that described in this manual may result in damage to the device performance.

If you make significant physical changes to a room, for instance visible structural changes or adding large items of furniture, this may affect the device performance and you should inform Oxehealth.

Maintenance

The Seclusion product analyses video from an infrared illuminated room that has been captured by a video camera. Hardware is often installed within a housing and so the cleanliness of the housing must be maintained. When cleaning, pay attention to the manufacturer's instructions for cleaning, and in particular, do not use solvent based, ammonia, caustic or abrasive cleaners or abrasive cleaning materials on the front, as this may scratch or fog the material, and potentially degrade the ability of the device to see the subject clearly and may reduce the reliability and accuracy of the output.

If components within the sealed housing are directly exposed to liquids, vapours or dust then they may be damaged and cease to operate correctly. If this happens, switch off the product and inform Oxehealth.

In the event of failure of any part of the Oxehealth product, inform Oxehealth.

Only Oxehealth or Oxehealth trained maintenance personnel may make modifications to this device as these may affect device performance. If you are in doubt whether changes could affect the device, you should contact Oxehealth who can advise.



IMPORTANT SAFETY AND SECURITY NOTICE - please read carefully

Maintaining the privacy of subjects, and protecting the confidentiality of their data is of the utmost importance. Ensure that you observe all of your organisation's safety, privacy and security policies and guidelines, and do not relocate any equipment that has been installed for the Oxehealth software, or change any security or access arrangements to the equipment without first informing Oxehealth.

If you are using any mobile devices to access your Oxehealth software, pay particular attention to ensuring these devices are used only by authorised and trained staff, using the devices as intended.

The Oxevision software is not intended to be used as a surveillance tool, and you must use the functions to view the video feed in accordance with your organisation's privacy and security policies. Failure to do so puts you and your organisation at risk of violating individuals' rights, and data protection laws.

How To Use the Seclusion Product

The Overview Screen

The **Overview Screen** shows all of the rooms connected to the Oxevision product, depending on your configuration. Seclusion rooms can be added to a dedicated device (see Figure 1) or added alongside devices showing standard rooms (see Figure 2).

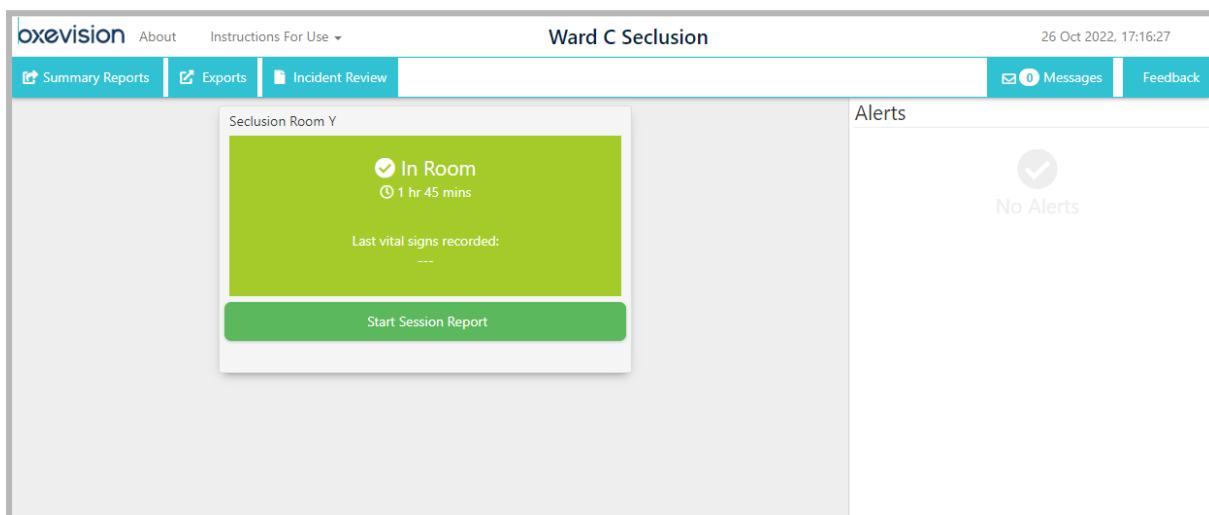


Figure 1 - Room Overview page, with a single Seclusion room tile on a dedicated device.


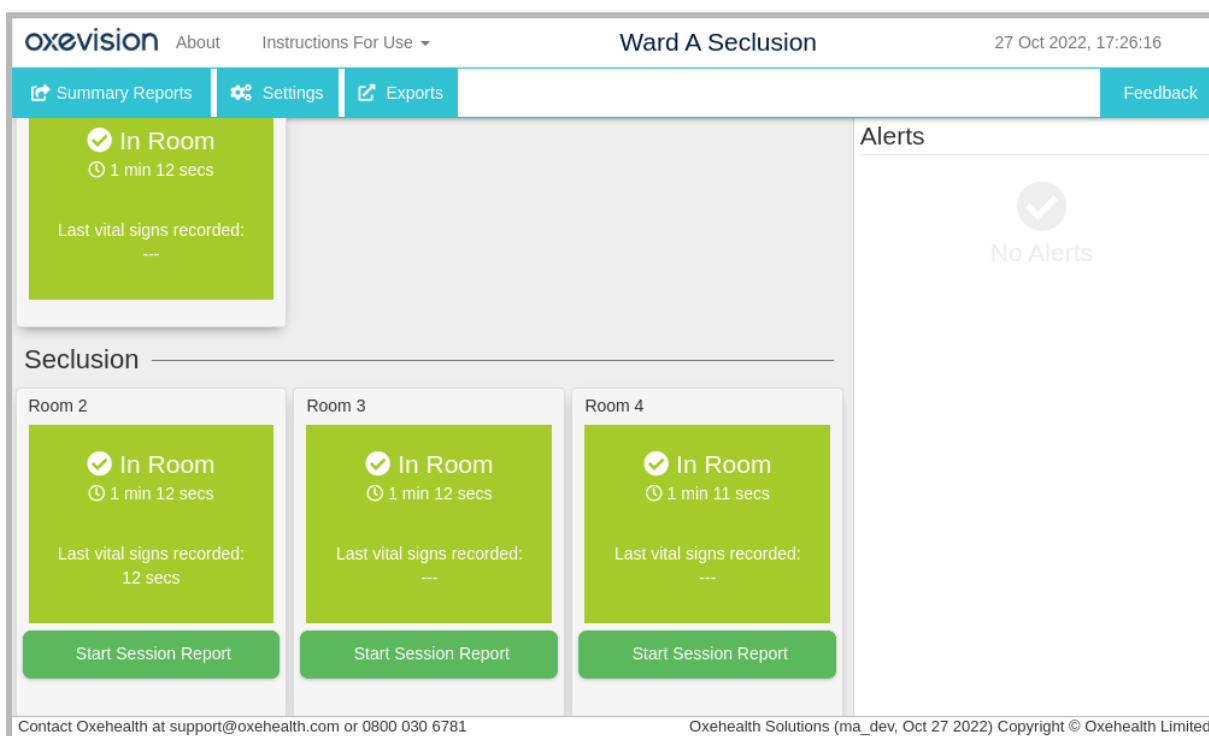



Figure 2 - Room Overview page on a shared device group, of standard rooms and seclusion rooms.

On the Overview Screen, the large icons indicate the status of the room as follows:


- **Disconnected:** the software is not connected to the room. No activity information will be generated.

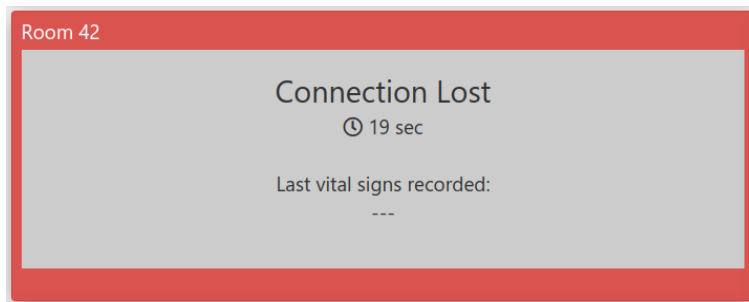


- **Connected:** the software is connected to the room and running, such that the software is receiving and analysing video. A green rectangle is shown, and the detected location of the occupant of the room is displayed as text. The time for which the room has been in that state is also displayed.

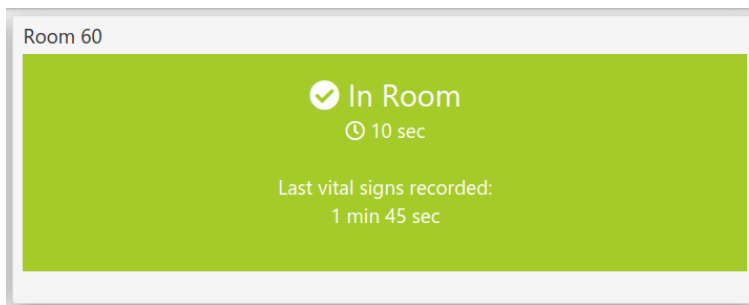

- **Sensor Off:** the software is connected to the room, but a user has chosen to stop the analysis software, such that the software has ceased receiving and analysing video for this room. No activity information will be generated.

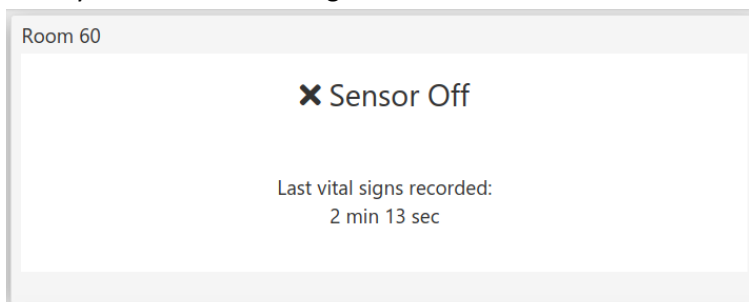

- **Alert State:** the software is connected to the room and running, such that the software is receiving video and analysing the video. In this state, other product modules are producing a non-medical alert to prompt further checks in the room. These prompts may be informational (orange), or alerts (red) as the examples below show, and will be pre-configured to a scheme specific to your installation.

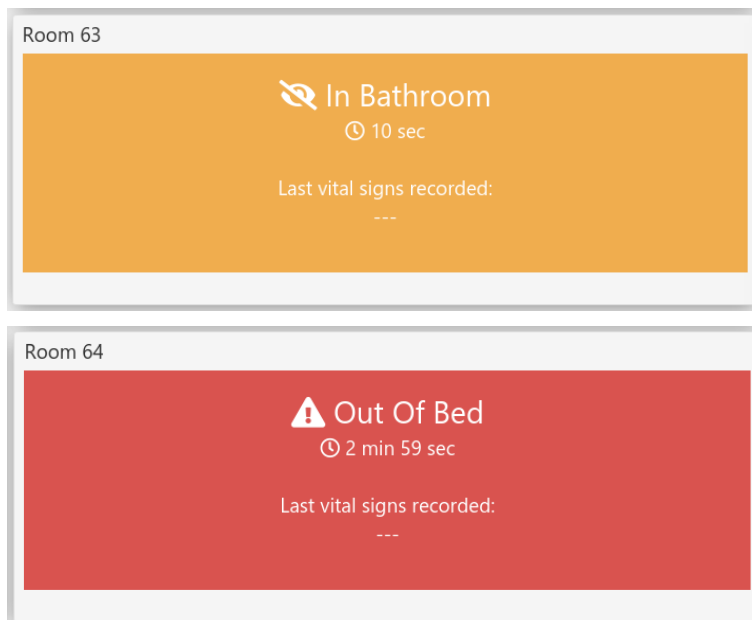

The text in the room icon will display the nature of the non-medical alert, and the time that the room has been in this state. Refer to other product module instructions for use for information on these alerts.

Starting a Session Report

The **Seclusion Overview Screen** will indicate the room status. If the room is connected (has good network connectivity), then you will be able to start the **Session Report**. To begin a session report click the **Start Session Report** button.

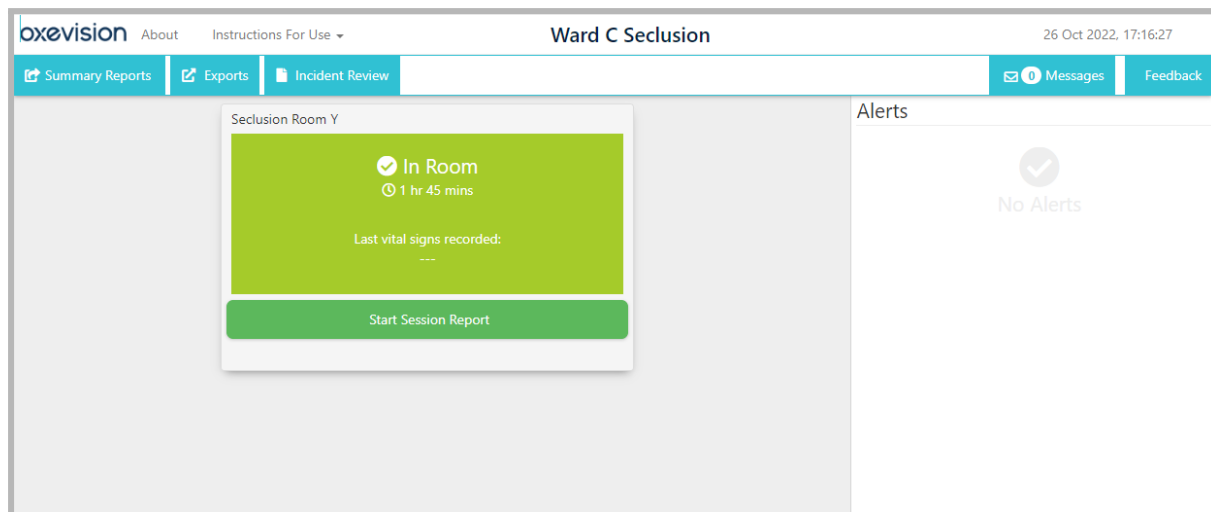


Figure 3 - Seclusion room card before a seclusion session has been started.

When the **Start Session Report** button has been clicked, the button on the room card will indicate the duration of time since the session was started (see Figure 5 below). Clicking the **Start Session Report** button again (now indicated by a timer displayed on a red button) will end the session.

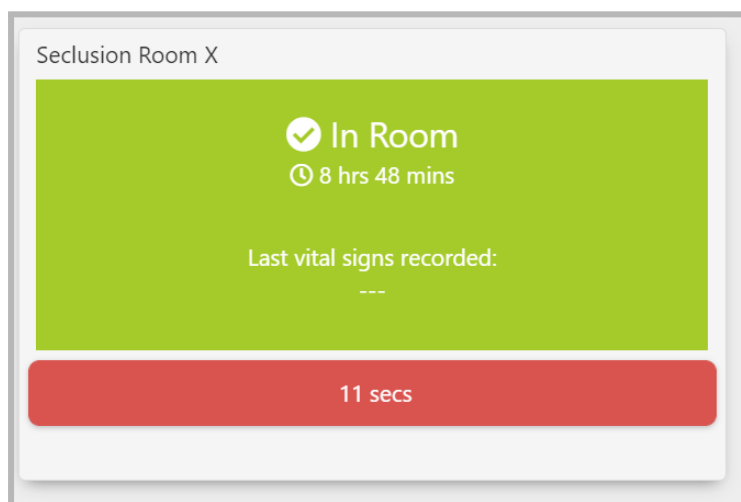


Figure 4 - Seclusion room card with a seclusion session in progress showing the time duration.

If there is a connection error when you attempt to start or end a seclusion session, the seclusion room card on the **Seclusion Overview Screen** will state an appropriate message of “**Failure To Start Session**” (see Figure 5a) or “**Failure To End Session**” (see Figure 5b).

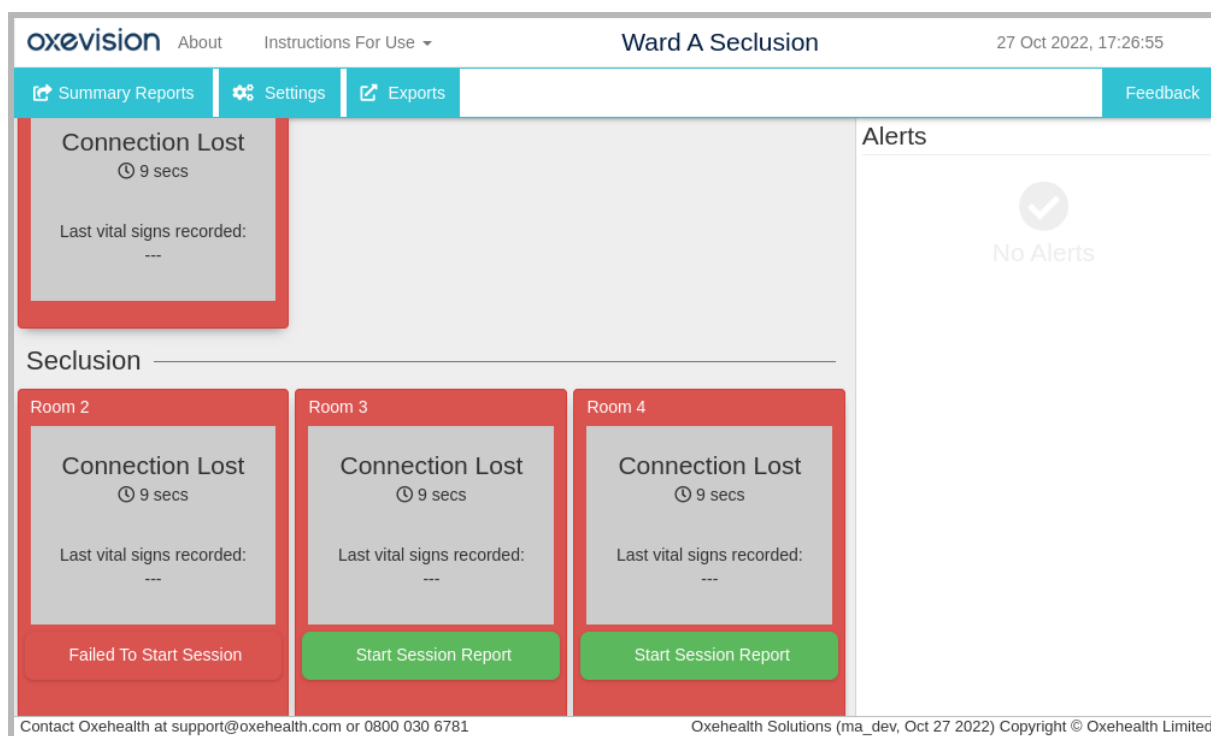


Figure 5a - The Room Overview page, on a single screen (device group), displaying seclusion room tiles with connection errors and the error message that occurs when attempting to start a session and it's unsuccessful

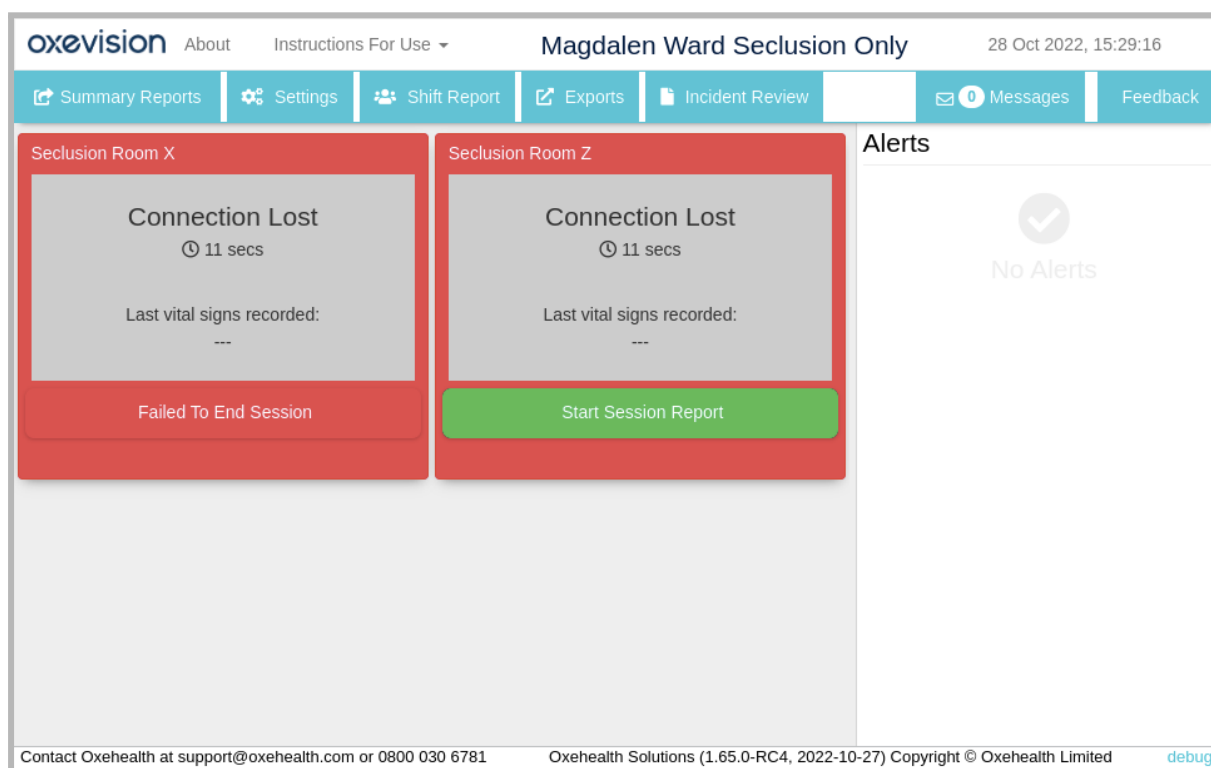


Figure 5b - The Room Overview page, on a single screen (non-shared device group), displaying seclusion room cards with connection errors and the error message that occurs when attempting to end a session and it's unsuccessful.

Taking Vital Signs Observations


For instructions on taking Vital Signs measurements, please refer to the **Vital Signs** instructions for use.

NOTE

Vital Signs readings taken prior to starting a Session Report will not be recorded in the Seclusion Session Report.

Viewing a Session Report

A **Session Report** can be viewed while a session is in progress by selecting the appropriate room card and clicking “**Session Report**” on the room **Action Menu**, as shown in Figure 6.

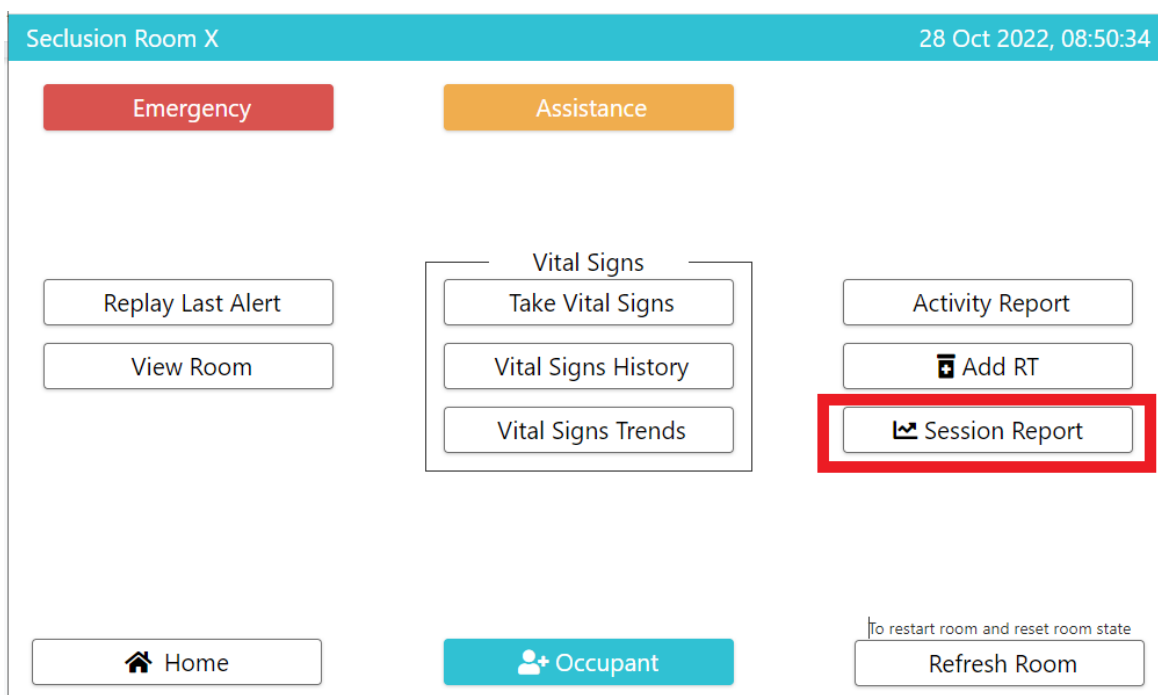

*Figure 6 - Room **Action Menu** with **Session Report** button highlighted. Availability of other buttons will vary depending upon your configuration.*

The **Session Report** will display a record of the patient’s **Vital Signs observations and trends**, **RT administration times** and patient **Activity** for each session as shown below in Figure 7.



*Figure 7 - Image of the **Session Report** after a session has been started.*

NOTE

The chart may show data for a subject who has been in the room but is not the subject currently occupying that room.

The user can scroll left and right to view across the Seclusion session timeline.

The patient's vital signs will be displayed on both the **Pulse Rate (PR)** and **Breathing Rate (BR)** chart with corresponding times, as shown in Figure 7.

- Any successful attempts will be shown with red dots (●) for Pulse Rate
- Blue dots (●) for breathing rate.
- All Vital Signs attempts (successful or unsuccessful) will be shown with a cross **X** indicating the times they were attempted.


Vital signs trends are shown with grey diamonds (◇). Details of Vital Sign Trends can be viewed in the Vital Sign IFU. The blue line on the **Activity Chart** indicates the Patient's activity level. The seclusion activity measure aims to give a summary indication of how active a patient in seclusion is - for example after rapid tranquilisation. The system looks over a time window (default 10 minutes) and builds a picture of the movement as:

- low level (fine movements, often not visible to the naked eye) which could be the small movements from breathing from someone lying down asleep or resting
- high level (large movements, visible to the naked eye) which is someone moving around the room

If available, **In Bathroom** periods, **Door Open** periods, **Alerts** and **Rapid Tranquilisation times (RT)** will also be displayed on the **Activity Chart** of the **Session Report** as shown in Figure 7.

Setting Patient ID

By clicking the **Patient ID** button, as seen in the bottom centre of Figure 7, the **patient's ID** can be added to the **Session Report** (Figure 8). Once the **Patient ID** has been entered, clicking **Save** or **Back** will navigate to the **Session Report** screen. Once a **Patient ID** is added, this can be edited or removed by clicking the **Patient ID** button once more, but only if the seclusion session is still active.

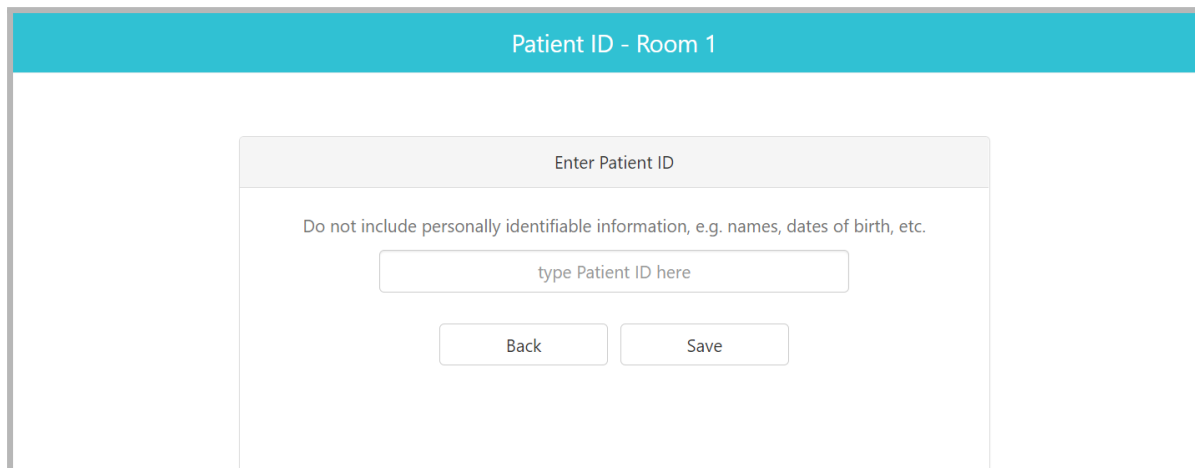

*Figure 8 - shows the **Patient ID** screen, where the **Patient ID** can be entered, edited and removed for a live **Session Report**.*



CAUTION

Do not include any personally identifying information in the Patient ID field.

Seclusion Summary Report

The Seclusion **Summary Report** shows the summary recorded details of the Seclusion Session Report, including **room states**, **rapid tranquillisation** summary and **timings**, **vital signs trend rates**, **vital signs attempts** and **completed vital signs observations**.

Seclusion **Summary Reports** can be accessed on a device where seclusion sessions are enabled by clicking the **Summary Reports** tab from the **Seclusion Overview Screen** (see Figure 9).

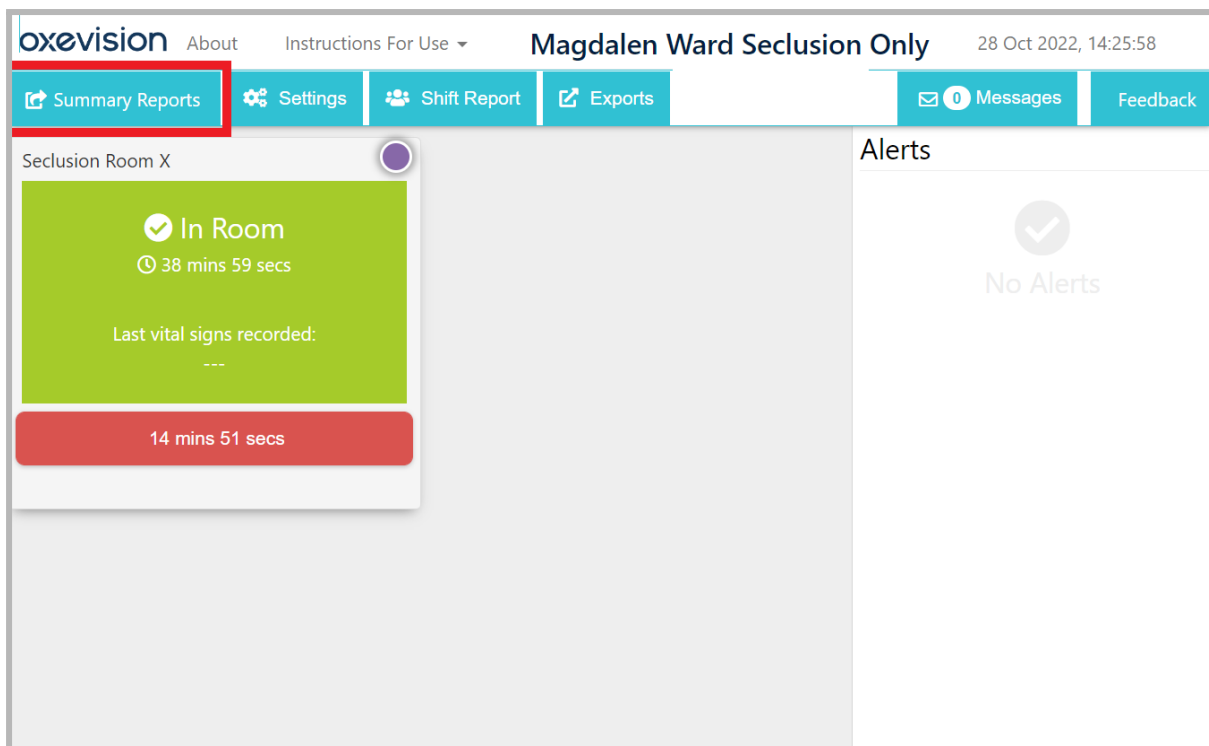



Figure 9 - Seclusion Overview Screen, with the Summary Reports tab highlighted.

The **Summary Reports** screen shows an archive of **Session Reports** in descending order over the previous seven days, for a maximum of two seclusion rooms (see Figure 10). An individual Seclusion **Summary Report** can be viewed by selecting the eye symbol  next to the corresponding report. An example **Summary Report** is shown in Figure 11.

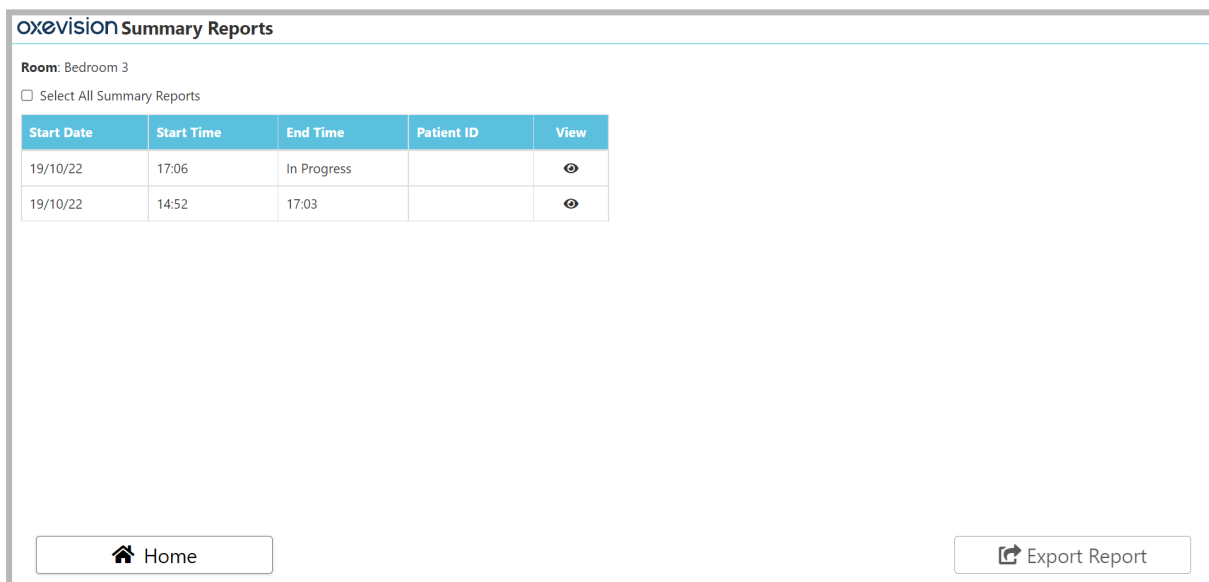


Figure 10 - Summary Reports screen.

OXeVISION Summary Report			
Patient ID : Room : Bedroom 3 Start Date : 19/10/22 Duration : 2 hrs 10 mins 41 secs Vital Signs : Attempted × 0, Vital Signs Taken × 0		RT : Recorded × 0 Door Open × 1 : 2 mins 4 secs In Room × 1 : 2 secs In Bathroom × 0 : 0 secs	
Date/Time	Events	BR	PR
19/10/22			
15:53:23	Empty (1 hr 16 mins 29 secs)	-	-
15:53:04	In Room (2 secs)	-	-
15:51:00	Door Open (2 mins 4 secs)	-	-
15:47:08	Empty (3 mins 52 secs)	-	-
15:47:07	Empty Alert (1 sec)	-	-
15:46:57	Empty (10 secs)	-	-
15:30:37	Empty (16 mins 4 secs)	-	-
15:08:15	Empty (22 mins 19 secs)	-	-
<input type="button" value="Home"/>		<input type="button" value="Summary Reports"/>	

Figure 11 - Example Summary Report.

When viewing a Seclusion Summary Report, users can return to the list of all available reports by clicking the **Summary Reports** button (see Figure 12).

OXeVISION Summary Report			
Patient ID : Room : Bedroom 3 Start Date : 19/10/22 Duration : 2 hrs 10 mins 41 secs Vital Signs : Attempted × 0, Vital Signs Taken × 0		RT : Recorded × 0 Door Open × 1 : 2 mins 4 secs In Room × 1 : 2 secs In Bathroom × 0 : 0 secs	
Date/Time	Events	BR	PR
19/10/22			
15:53:23	Empty (1 hr 16 mins 29 secs)	-	-
15:53:04	In Room (2 secs)	-	-
15:51:00	Door Open (2 mins 4 secs)	-	-
15:47:08	Empty (3 mins 52 secs)	-	-
15:47:07	Empty Alert (1 sec)	-	-
15:46:57	Empty (10 secs)	-	-
15:30:37	Empty (16 mins 4 secs)	-	-
15:08:15	Empty (22 mins 19 secs)	-	-
<input type="button" value="Home"/>		<input type="button" value="Summary Reports"/>	

Figure 12 - Selecting Summary Reports will return to view the list of all available seclusion summary reports as shown in Figure 10.

A Seclusion Summary Report can also be accessed from the Seclusion Session Report screen while a session is in progress. Clicking the **Export Report** button (Figure 13) on the Session Report screen will display the Summary Report.

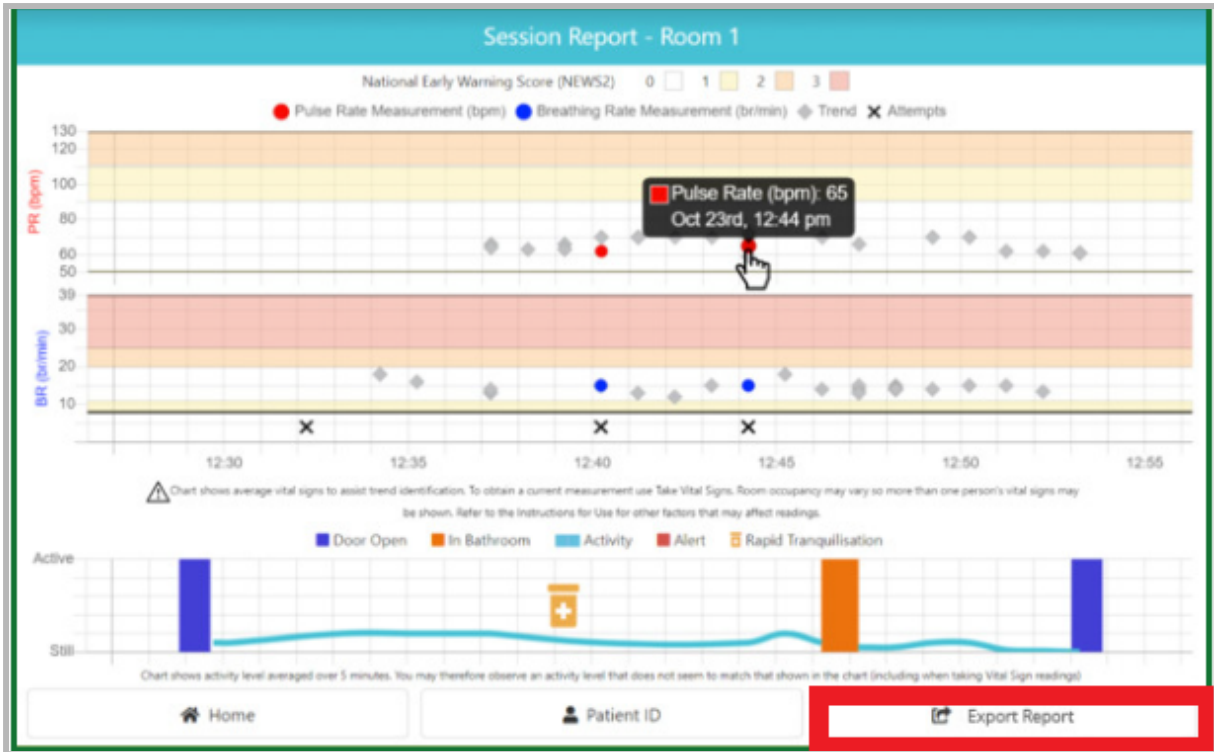


Figure 13 - Selecting **Export Report** on a Session Report will open the Summary Report.

When viewing the Seclusion Summary Report via the Seclusion Session Report page, users may navigate to **Home** (to the Overview Screen) or the Export Report screen (see Figure 14).

OXEVISION Summary Report

Patient ID : RT : Recorded x 0
 Room : Bedroom 3 Door Open x 1 : 2 mins 4 secs
 Start Date : 19/10/22 In Room x 1 : 2 secs
 Duration : 2 hrs 10 mins 41 secs In Bathroom x 0 : 0 secs
 Vital Signs : Attempted x 0, Vital Signs Taken x 0

Date/Time	Events	BR	PR
19/10/22			
15:53:23	Empty (1 hr 16 mins 29 secs)	-	-
15:53:04	In Room (2 secs)	-	-
15:51:00	Door Open (2 mins 4 secs)	-	-
15:47:08	Empty (3 mins 52 secs)	-	-
15:47:07	Empty Alert (1 sec)	-	-
15:46:57	Empty (10 secs)	-	-
15:30:37	Empty (16 mins 4 secs)	-	-
15:08:15	Empty (22 mins 19 secs)	-	-

Home Summary Reports

Figure 14 - Seclusion Summary Report viewed via Seclusion Session Report screen.

Exporting a Summary Report

Exporting via the Summary Reports screen

A Seclusion Summary Report can be exported either via the Summary Reports screen or via the Seclusion Session Report screen for a specific seclusion session.

Selecting the **Summary Reports** tab on the Seclusion Overview screen will open the Summary Reports screen where a list of all the reports for a maximum of two seclusion rooms can be viewed (see Figure 15). The seclusion summary reports are retained in this list for seven days.

The screenshot shows the 'OXVISION Summary Reports' interface. It is divided into two panels for 'Room: Staging Room' and 'Room: Room 2'. Each panel has a 'Select All Summary Reports' checkbox and a table of reports. The 'Staging Room' table has one row with Start Date 20/10/22, Start Time 14:22, and End Time 14:22. The 'Room 2' table has one row with Start Date 20/10/22, Start Time 14:22, and End Time In Progress. Both rows have a 'View' icon. At the bottom, there are 'Home' and 'Export Report' buttons.

Room: Staging Room					Room: Room 2				
Start Date	Start Time	End Time	Patient ID	View	Start Date	Start Time	End Time	Patient ID	View
20/10/22	14:22	14:22		👁️	20/10/22	14:22	In Progress		👁️

Figure 15: Summary Reports screen, showing two seclusion rooms with visible reports available from the previous seven days.

One or more reports can be selected by clicking the relevant row(s) in the table (see Figure 16). Selected seclusion summary reports are highlighted with green shading and can be deselected by clicking the relevant row again in the table.

The screenshot shows the 'OXVISION Summary Reports' interface with two reports selected. The 'Staging Room' table has two rows: one with Start Date 20/10/22, Start Time 14:32, and End Time In Progress; and another with Start Date 20/10/22, Start Time 14:22, and End Time 14:22. The 'Room 2' table has one row with Start Date 20/10/22, Start Time 14:22, and End Time In Progress. All three rows are highlighted in green and have a 'View' icon. At the bottom, there are 'Home' and 'Export Report' buttons.

Room: Staging Room					Room: Room 2				
Start Date	Start Time	End Time	Patient ID	View	Start Date	Start Time	End Time	Patient ID	View
20/10/22	14:32	In Progress		👁️	20/10/22	14:22	In Progress		👁️
20/10/22	14:22	14:22		👁️					

Figure 16 - Summary Reports screen showing multiple selected seclusion summary reports.

All summary reports for a particular room can be selected by checking the relevant Select All Summary Reports checkbox (see Figure 17).

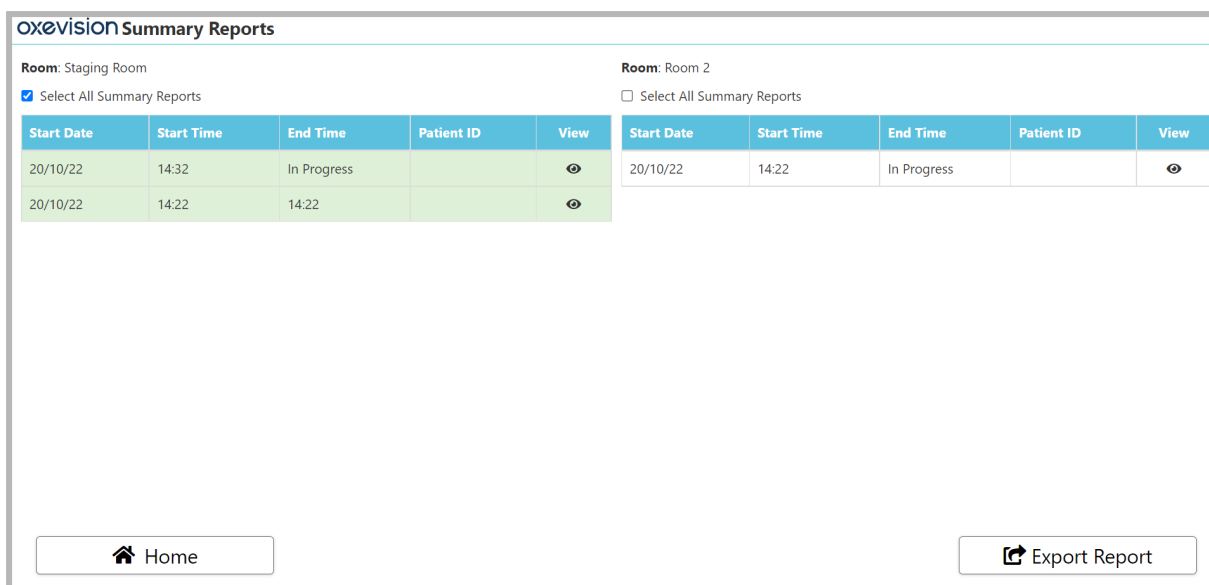


Figure 17 - Summary Reports screen with the Select All Summary Reports checkbox selected.

Once the intended Seclusion Summary Report(s) have been selected, clicking the Export Report button will navigate the user to the Export Summary Report screen (see Figure 18). Both the Email and Reason For Export fields must be completed before the Confirm Export button is clicked in order for the export to be successful. The Email field must contain a valid email address and the Reason for Export field must contain a text entry.

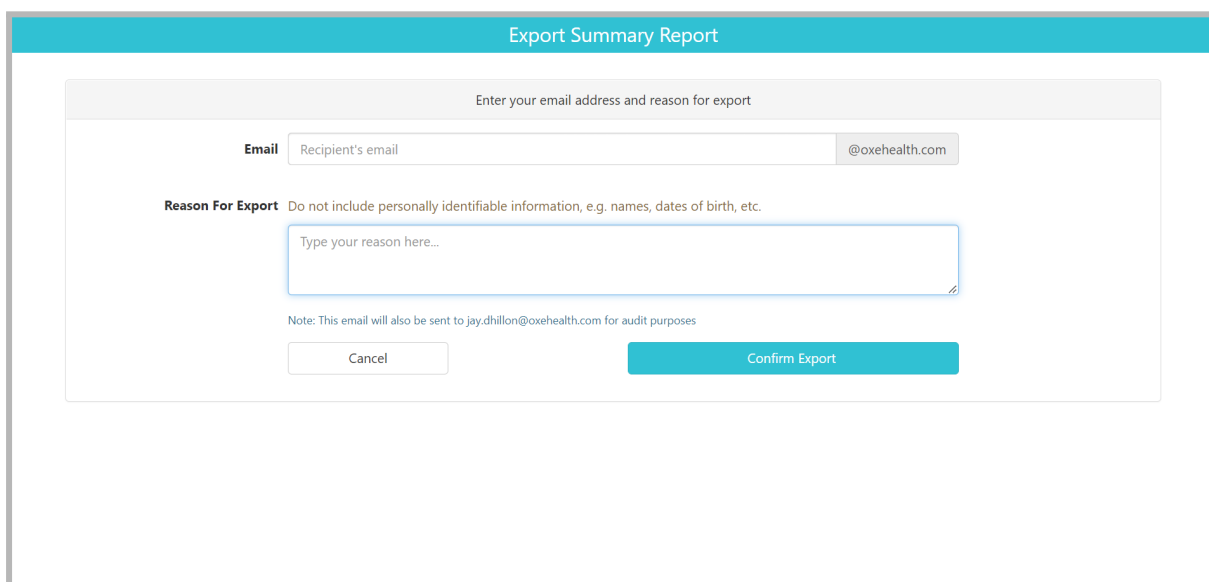


Figure 18 - Export Summary Page to export the Seclusion Summary Report.

Exporting via a Session Report

While a seclusion session is in progress, the corresponding Seclusion Summary Report can be exported by first clicking **Export Report** on the Session Report screen (see Figure 19), and then clicking **Export Report** on the Summary Report screen (see Figure 20).

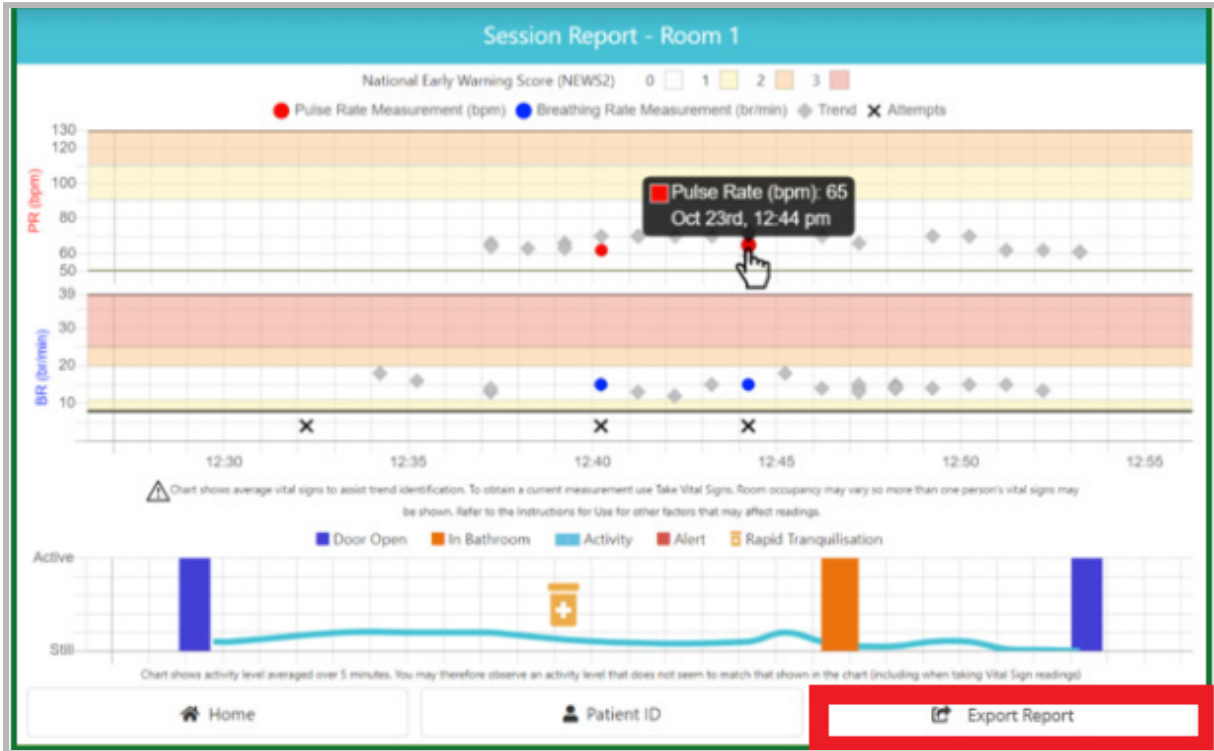


Figure 19 - Export Report feature on the Seclusion Session Report screen.



Figure 20 - Export Report feature on the Seclusion Summary Report screen.

Clicking the **Export Report** button on the Summary Report screen will navigate to the Export Summary Report screen (see Figure 21). Both the Email and Reason for Export fields must be completed correctly before the Confirm Export button is clicked in order for the export to be

successful. The **Email** field must contain a valid email address and the **Reason for Export** field must contain a text entry.

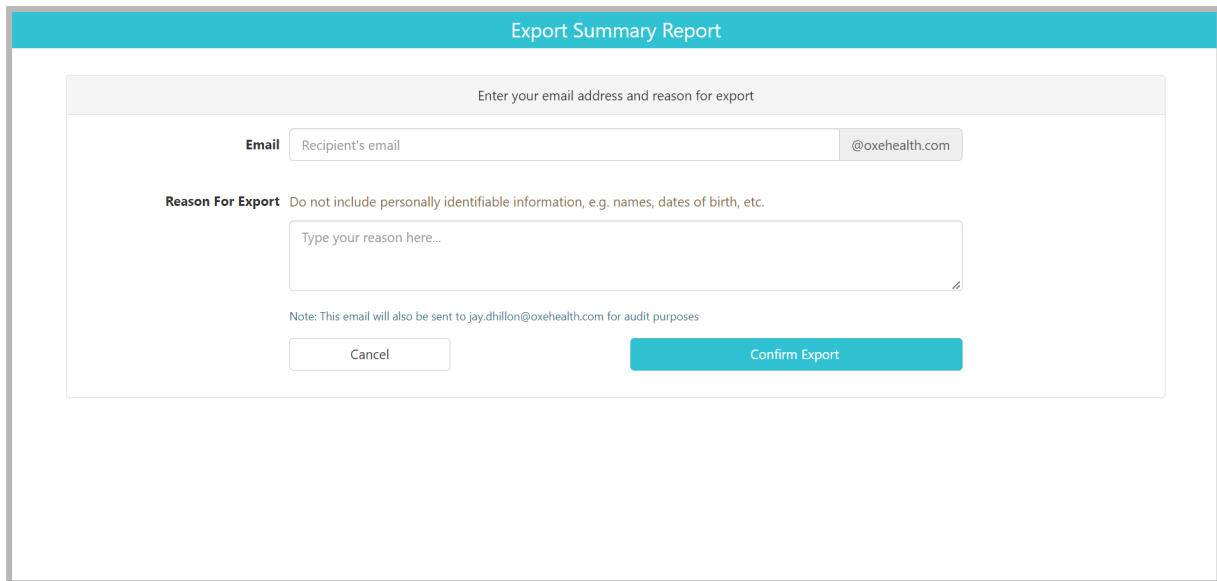


Figure 21 - Export Summary screen to export the Seclusion Summary Report.

Clicking Confirm Export will send a PDF of the Seclusion Summary Report to the entered recipient's email and any default recipients specified for that ward.

Room Actions for Seclusion Rooms

The following room **Action Menu** features are available for seclusion rooms as shown below in Figure 22:

- Seclusion Room with Seclusion Session Report enabled only*:
 - Add RT
 - Session Report
 - Patient ID
 - Take Vital Signs (part of the Vital Signs medical device)
 - Vital Signs Trends (part of the Vital Signs medical device)

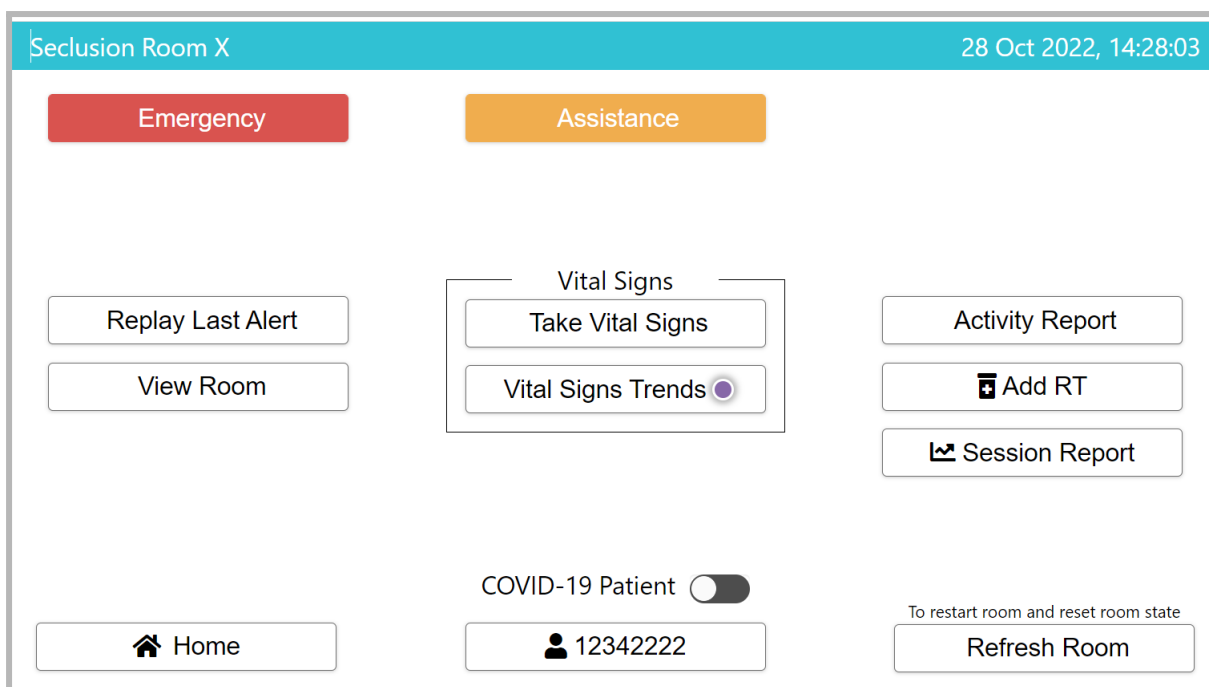


Figure 22 - Room Action Menu screen for a seclusion room where all the possible room Menu Actions are configured to be available, except Pause Room, which is disabled by default for a seclusion room with the Seclusion Session Report enabled.

The room Action Menu may include other menu items for other product modules (refer to the other Oxehealth product instructions for use for more information).

Refer to the Oxehealth Vital Signs instructions for use for more information on how to use Take **Vital Signs** and **Vital Signs Trends**.

Patient ID

A patient's ID can only be added once a seclusion session has commenced. Clicking on **Patient ID**, as seen in the bottom centre of Figure 22, will enable you to add the patient's ID details (see Figure 23). Once the Patient ID has been entered, clicking Save or Back will return to the Room Action Menu screen. **Note:** we recommend that no patient-identifiable information should be entered in this field, for example, names, dates of birth, etc. The Enter Patient ID input only permits the numbers 0-9 and "." (full stops).

Once a Patient ID is added, this can be edited or removed subsequently if the seclusion session is still active by clicking the Patient ID field once more.

Figure 23 - shows the Patient ID screen, which enables the ability to set, edit and remove a Patient ID for a given seclusion session.

There is also the ability to add the Patient ID from the Seclusion Session Report itself; please refer to that section for details.

If a Patient ID is set and not removed prior to the seclusion session ending, then this is saved with the Seclusion Summary Report and is viewable on the Summary Reports screen (see Figure 24).

Room: Seclusion Room X					Room: Seclusion Room Z				
<input type="checkbox"/> Select All Summary Reports					<input type="checkbox"/> Select All Summary Reports				
Start Date	Start Time	End Time	Patient ID	View	Start Date	Start Time	End Time	Patient ID	View
28/10/22	14:11	In Progress	12342222	👁	01/11/22	08:49	In Progress		👁
					30/10/22	11:36	08:49	123	👁
					28/10/22	15:28	15:28		👁
					28/10/22	15:28	15:28		👁
					28/10/22	15:28	15:28		👁
					28/10/22	15:28	15:28		👁
					28/10/22	15:28	15:28		👁
					28/10/22	15:28	15:28		👁
					28/10/22	15:28	15:28		👁
					28/10/22	15:28	15:28		👁

Figure 24 - Summary Reports screen with a populated Patient ID highlighted.

Tracking when Rapid Tranquillisation is administered

In the case where Rapid Tranquillisation (RT) is administered, the **+ Add RT** button allows users to record instances of rapid tranquillisation which will then be included in the respective Seclusion Session Report and Seclusion Summary Report. Clicking **+ Add RT** on the room **Action Menu** screen (see Figure 25) for the selected room opens the **Add RT** screen (see Figure 26).

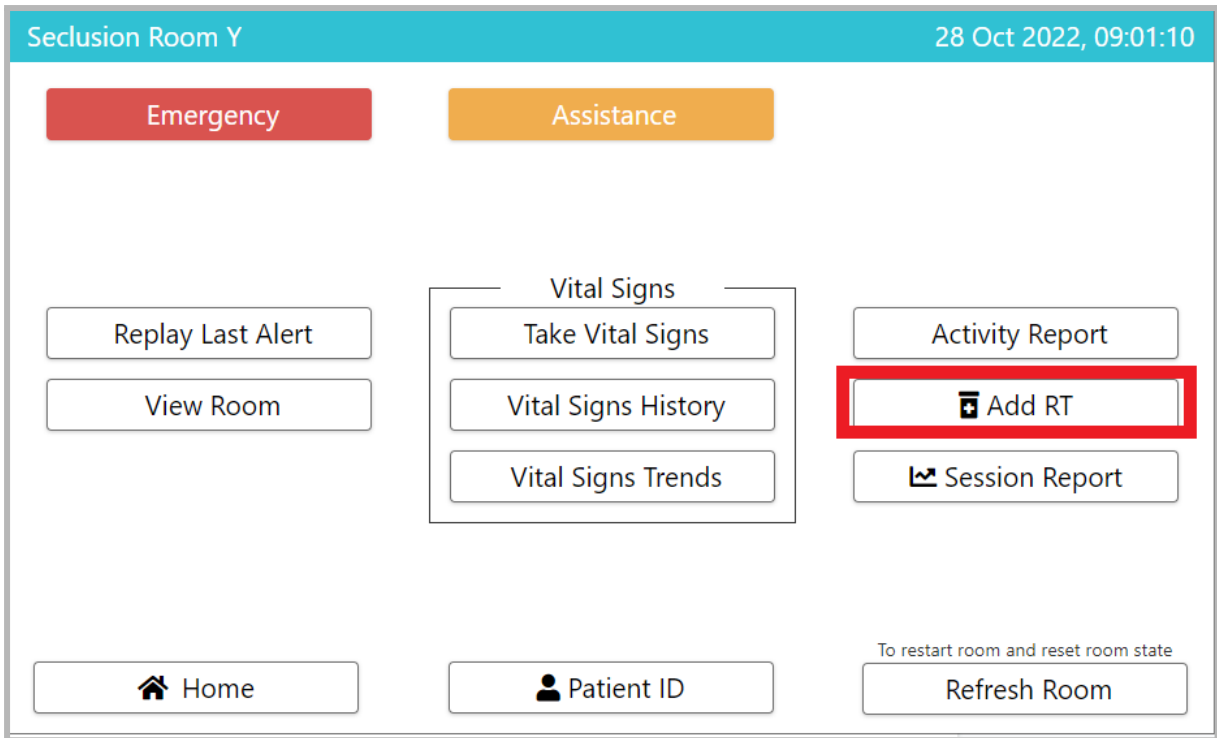


Figure 25 - Room Action Menu, with the + Add RT button highlighted.


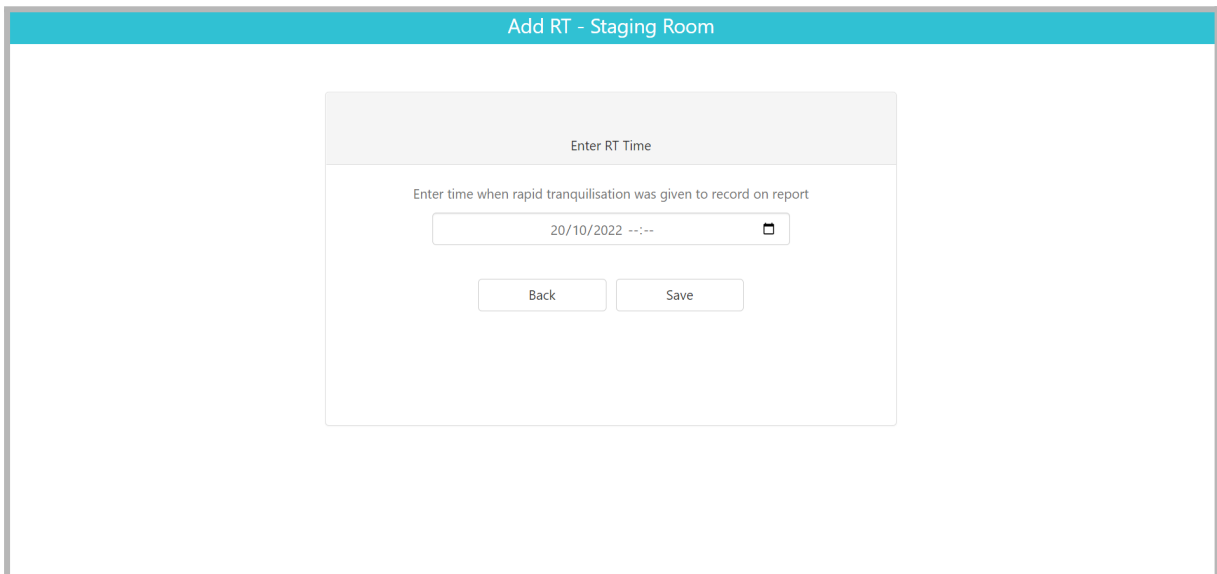


Figure 26 - The Add RT screen to enable the recording of rapid tranquilisation events.

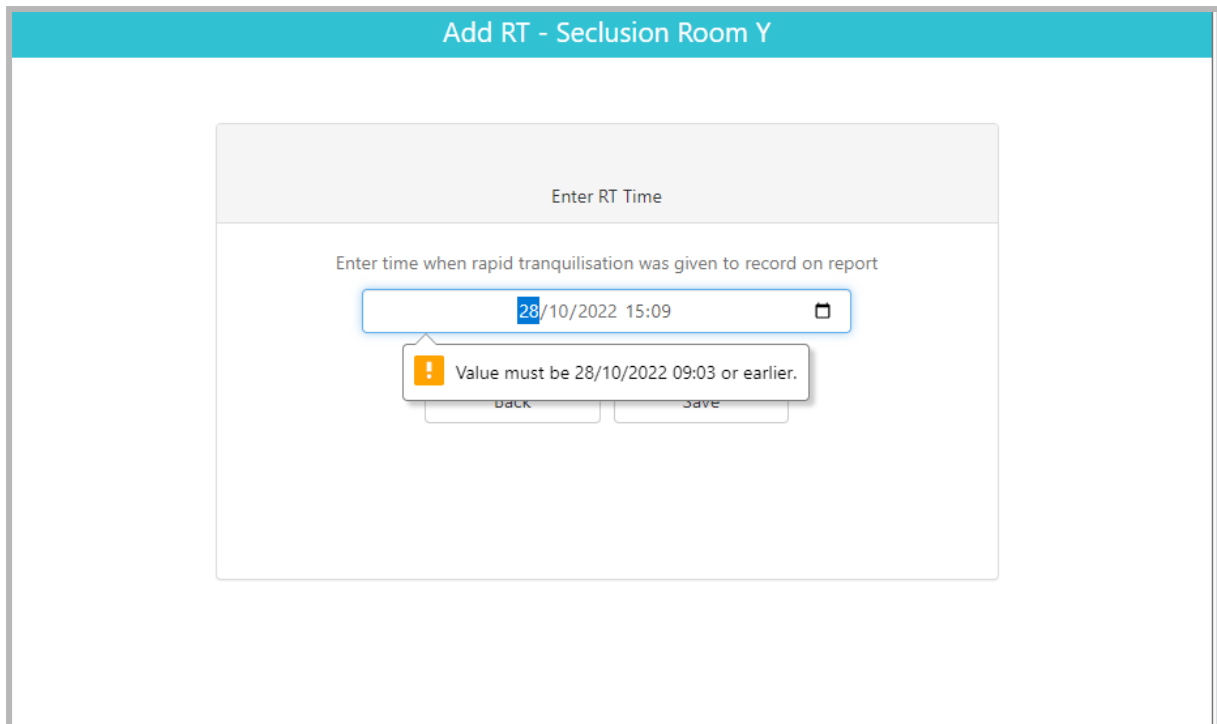

The screenshot shows a web interface for adding a Rapid Tranquilisation (RT) event. The title bar at the top is teal and contains the text "Add RT - Seclusion Room Y". Below this is a white form area with a grey header "Enter RT Time". The main content of the form is a text input field with the placeholder text "Enter time when rapid tranquilisation was given to record on report". The input field contains the value "28/10/2022 15:09" and has a calendar icon on the right. Below the input field, a yellow warning box with an exclamation mark icon displays the message "Value must be 28/10/2022 09:03 or earlier." At the bottom of the form, there are two buttons: "Back" and "Save".

Figure 27 - The + Add RT screen with date validation.

Rapid tranquilisation events cannot be retrospectively entered prior to the start of a seclusion session, and events cannot have a time set in the future. If the start of the session time is selected, the system will automatically add 5 seconds to this time, so that it can be visible on the Seclusion Session Report. Figure 28a shows an example of how rapid tranquilisation events are displayed on a Seclusion Session Report and Figure 28b shows an example of how rapid tranquilisation events are displayed on a Seclusion Summary Report.

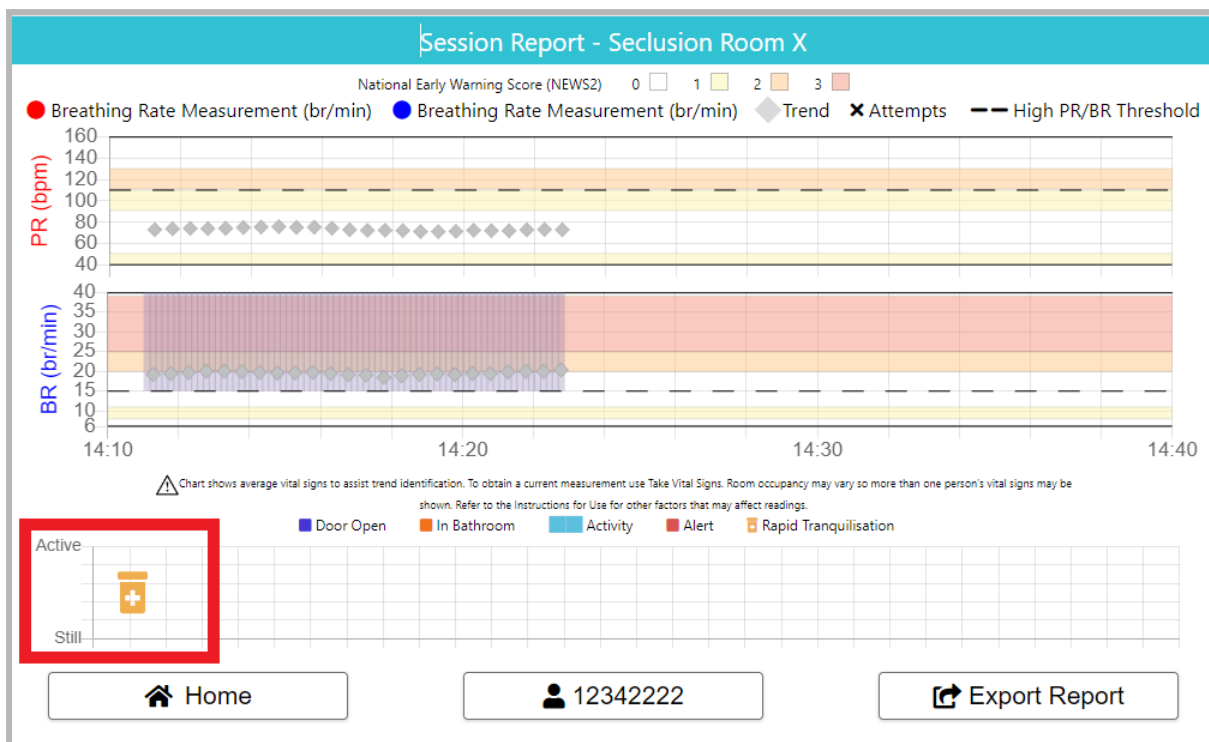


Figure 28a - an example of **rapid tranquilisation** events highlighted on a Seclusion Session Report.

oxe vision Summary Report

Patient ID : 12342222 RT : Recorded x 1
 Room : Seclusion Room X Door Open x 0 : 0 secs
 Start Date : 28/10/22 In Room x 0 : 0 secs
 Duration : 22 mins 5 secs In Bathroom x 0 : 0 secs
 Vital Signs : Attempted x 1, Vital Signs Taken x 0

Date/Time	Events	BR	PR
28/10/22			
14:11:46	BR High (23 mins 13 secs)	-	-
14:11:05	RT Administered	-	-

Home Export Report

Figure 28b - an example of **rapid tranquilisation** events highlighted on a Seclusion Summary Report.

WARNING

The Seclusion product is not a medication record or planning tool for tranquilisation of patients. Decisions or recommendations for administering tranquilisation must not be made based on the rapid tranquilisation indications in the Oxehealth Seclusion session report.

Click **Home** to return to the **Seclusion Overview Screen**.

Prompts for Vital Signs Overdue

When vital signs observations are due to be taken, a visual prompt will be displayed to remind you to take their vital signs. The timing of this prompt may be different for each seclusion room and is configured as part of your installation..

This prompt is displayed as a notification banner over all screens showing the seclusion room unless vital signs are already being taken as shown in Figure 29 below.

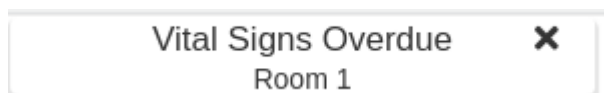


Figure 29 - The Vital Signs Overdue Prompt on a Seclusion device.

The prompt may be dismissed and closed by selecting **X** or swiping the prompt left or right. The prompt will be automatically removed if you take vital signs using the **Take Vital Signs** feature.

When you (or someone else) takes vital signs observations using the **Take Vital Signs** feature, the timer will restart the timer, and dismiss any **Vital Signs Overdue** prompts visible on other devices displaying the seclusion room.

NOTE

The vital sign observation protocol is subject to your organisation's vital signs observation protocol which may indicate the time required between each observation. For instance, if vital sign observations should be taken at 15-minute intervals the prompt can be used to help remind the staff that the observation is due. This time period will be configured as part of your installation.

The Oxehealth Seclusion product should not be relied upon as a clinical tool for adherence to observation protocols, and this overdue prompt is provided only as a reminder to staff.

Sending Feedback to Oxehealth

To send feedback of any sort to Oxehealth, click the **Feedback** button in the top right corner of the screen. You will then see the **Feedback screen** below:

Feedback
21 Jul 2022, 14:04:42

Name

Contact number

Feedback Type

General query
▼

Do not include personally identifiable information, e.g. names, dates of birth, etc.

Message

Enter your message here

🏠 Home

Send Feedback

Select the **Feedback Type** (if this is available) and fill in the **Message** text field with the information you would like to share, your issues, suggestions or questions and click **Send Feedback** to send the information to Oxehealth. Please also provide your name and contact details, if available, as this will help a support representative to contact you if you require assistance. We will then receive your message automatically, and you will be returned to the **Seclusion Overview screen**.

If you do not wish to send feedback, click **Cancel** and you will be returned to the **Overview screen**.



CAUTION

Do not include any subject identifying information in the feedback, e.g. names, date of births.

Manufacturer



Oxehealth Limited

Magdalen Centre North
Oxford Science Park
Oxford
OX4 4GA
United Kingdom

Telephone: +44(0)800 030 6781

Website: www.oxehealth.com

Email: support@oxehealth.com